




Patient terminal PAT

User manual

HEALTH CARE

SCHRACK
S E C O N E T

Overview

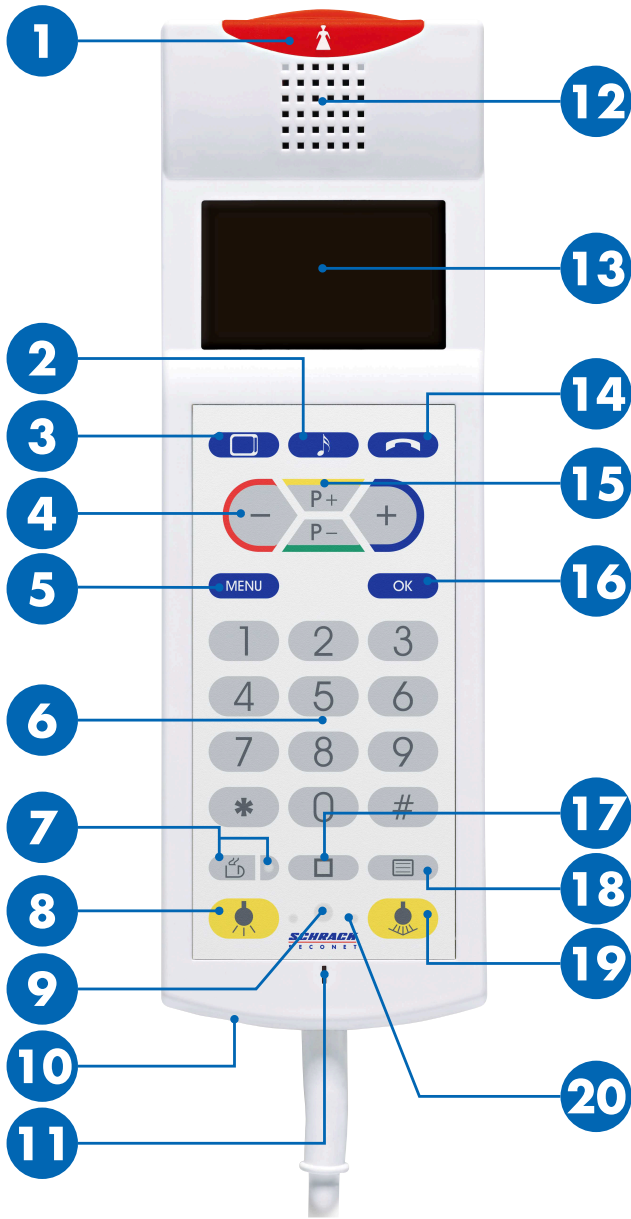

Care staff call

The call button  1 is constantly lit, so that it is easier to locate even in the dark.

Call care staff

Press button  1.

► The call button  1 lights up.

Care staff answers

1. Gong sounds over loudspeaker 12 and the call button flashes.
2. Use the patient terminal like a telephone for the talk with the care staff.
3. The care staff ends the call at the end of the talk.

Service call

The LED of the button is constantly lit, so that it is easier to locate even in the dark.

Press service call button  7.

► The service call button LED  7 lights up.

The service staff will either call you on the device or will come to you in person.

Turn light on and off

The LED of the light buttons 20 are constantly lit, so that the buttons are easier to locate even in the dark.

Room light (optional)

Press the room light button  8 briefly.

► Room light is turned on or off.

Reading light (optional)

Press the reading light button  19 briefly.

► Reading light is turned on or off.

Menu functions

Set display brightness

1. Press button **MENU** 5.
2. Select **Brightness** on the display 13 using the program selection buttons **P+ / P-** 15.
3. Confirm with **OK** button 16.
4. Set the desired brightness using **- / +** buttons 4.
5. Using the menu button **MENU** 5 to leave menu.

Set display contrast

1. Press button **MENU** 5.
2. Select **Contrast** on the display 13 using the program selection buttons **P+ / P-** 15.
3. Confirm with **OK** button 16.
4. Set the desired contrast using **- / +** buttons 4.
5. Using the menu button **MENU** 5 to leave menu.

Set volume

1. Press button **MENU** 5.
2. Select **Volume** on the display 13 using the program selection buttons **P+ / P-** 15.
3. Confirm with **OK** button 16.
4. Select for which function the volume should be set using the program selection buttons **P+ / P-** 15.
5. Set the desired volume using **- / +** buttons 4.
6. Using the menu button **MENU** 5 to leave menu.

Set ringtone

1. Press button **MENU** 5.
2. Select **Ring tones** on the display 13 using the program selection buttons **P+ / P-** 15.
3. Confirm with **OK** button 16.
4. Set the desired ring tone using the program selection buttons **P+ / P-** 15.
5. Using the menu button **MENU** 5 to leave menu.

Optional functions

TV

Turn TV set on and off

Press TV button  3 to turn the TV set on or off.

- ▶ The TV set is turned on and the last channel watched will be indicated on the display 13.

Choosing a channel


Enter the channel number using the numerical keypad 6 or switch to the next channel using the program selection buttons **P+ / P-** 15.

Setting sound and volume

Sound output via the loudspeaker 12 or connected headphones (connection socket 10).

Set the desired volume using **- / +** buttons 4.

Call up teletext

1. Press teletext button  18 to call up or leave teletext mode.
2. Enter the page number using the numerical keypad 6 or switch to the next page using the program selection buttons **P+ / P-** 15.

Radio

Turn radio on and off

Press radio button  2 to turn the radio on or off.

- ▶ The radio is turned on and the last radio channel listened to will be indicated on the display 13.

Choosing a channel

Enter the channel number using the numerical keypad 6 or switch to the next channel using the program selection buttons **P+ / P-** 15.



Setting sound and volume

Sound output via the loudspeaker 12 or connected headphones (connection socket 10).



Set the desired volume using **- / +** buttons 4.

Telephone



Accept a call

1. Accept an incoming call using the telephone button  14.
2. Set the desired volume using **- / +** buttons 4.
3. Press the telephone button  14 to end the call after the conversation.


Making a telephone call

1. Press the telephone button  14.
2. Press numerical key for trunk line/public switched telephone network (usually 0, please check with the staff).
3. Call the desired telephone number using the numerical keypad 6.
4. Confirm with **OK** button 16.
5. Press the telephone button  14 to end the call after the conversation.

Last number redial

1. Press the telephone button  14.
2. The last ten dialled telephone numbers are indicated on the display 13.
3. Select the desired number using **- / +** buttons 4.
4. Confirm with **OK** button 16.
5. Press the telephone button  14 to end the call after the conversation.

Blinds control

1. Press special functions button  17.
2. Select **Blinds** on the display 13 using the program selection buttons **P+ / P-** 15.
3. Open or close the blinds using the **- / +** buttons 4.

Infrared receiver

The infrared receiver is used to remotely control the patient terminal via environmental control devices.

1	Call button
2	Radio button
3	TV button
4	-/+ button
5	Menu button
6	Numerical keypad
7	Service call button with LED (finder light)
8	Room light
9	Infrared receiver (optional)
10	Headphone connection
11	Microphone
12	Loudspeaker
13	Display
14	Telephone button
15	Program selection buttons
16	Confirmation button
17	Special function button
18	Teletext button
19	Reading light
20	LED of light button (finder light)



CAUTION

Do not use as rising aid

Cable and device are no rising aid. Do not pull yourself up using the device.



SCHRACK SECONET AG
Eibesbrunnnergasse 18
A-1120 Vienna
Tel. +43 1 81157
office@schrack-seconet.com
www.schrack-seconet.com

EN

HEALTH CARE

SCHRACK
SECONET