


# Pear pushbutton BT-B

---

**User manual**

# Overview

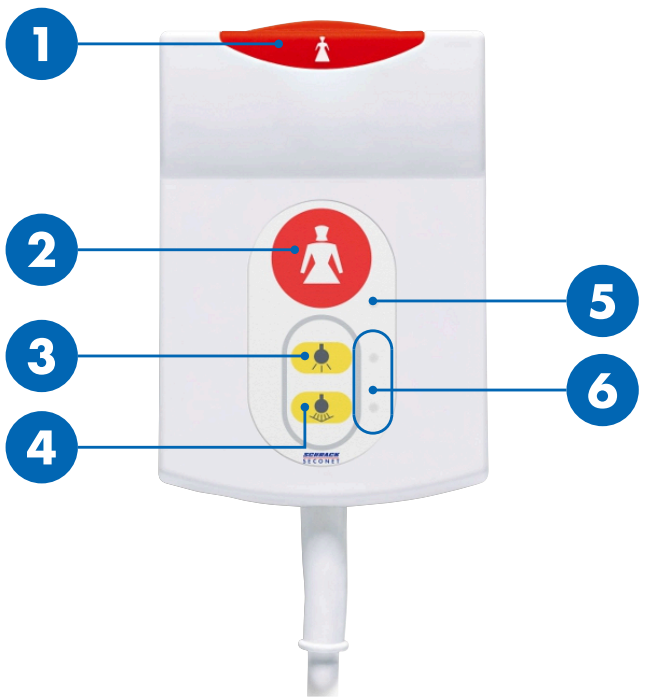

- 1 / 2 Call buttons
- 3 Room light
- 4 Reading light
- 5 LED of call button (reassurance light)
- 6 LED of light button (finder light)




#### **CAUTION**

##### **Do not use as rising aid**

Cable and device are no rising aid. Do not pull yourself up using the device.

## Care staff call

The call button  1 and the LED 5 are constantly lit, so that the buttons are easier to locate even in the dark.

### Call care staff

Press call button  1 or 2.

- ▶ Call button  1 and LED 5 light up.

## Turn light on and off

The LED of the light buttons 6 are constantly lit, so that the buttons are easier to locate even in the dark.

### Room light (optional)

Press the room light button  3 briefly.

- ▶ Room light is turned on or off.

### Reading light (optional)

Press the reading light button  4 briefly.

- ▶ Reading light is turned on or off.

## **Schrack Seconet AG**

Eibesbrunnnergasse 18 | A-1120 Vienna

+43 50 857 | [office@schrack-seconet.com](mailto:office@schrack-seconet.com)

## **[schrack-seconet.com](https://www.schrack-seconet.com)**

Czech Rep., CZ-149 00 Prague 4, Štřitová 283 | +420 2 74784422

Hungary, HU-1119 Budapest, Fehérvári út 89-95 | +36 1 4644300

India, IN-122102 Gurgaon, C-704A, Pioneer Urban Square, Sec-62 | +91 124 4141501

Poland, PL-02-972 Warsaw, ul. Branickiego 15, Wilanów Office Park, bud. B1 | +48 22 3300620

Romania, RO-023961 București, Str. Mântuleasa nr. 15A/1 | +40 372 756316

Russia, RU-123001 Moscow, B. Sadovaya str. 5, build. 1 office 514 | +7 495 5105015

Slovakia, SK-831 06 Bratislava, Mudrochova 2 | +421 2 44635595

Sweden, SE-126 30 Hägersten, Vretenborgsvägen 28, Floor 9 | +46 8 6801860

Turkey, TR-34718 Kadıköy-İstanbul, Koşuyolu Mah. İsmailpaşa Sk.No: 78 | +90 216 3455199