



# Patient terminal PAT-E

**User manual**

HEALTH CARE

**SCHRACK**  
S E C O N E T



## Care staff call

The call button  **1** is constantly lit, so that it is easier to locate even in the dark.

### Call care staff

Press call button  **1** or **2**.

► The call button  **1** lights up.

### Care staff answers

1. Gong sounds over loudspeaker **11** and the call button flashes.
2. Use the patient terminal like a telephone for the talk with the care staff.
3. The care staff ends the call at the end of the talk.

## Service call

The LED of the button is constantly lit, so that it is easier to locate even in the dark.

Press service call button  **6**.

► The service call button LED  **6** lights up.

The service staff will either call you on the device or will come to you in person.

## Turn light on and off


The LED of the light buttons **19** are constantly lit, so that the buttons are easier to locate even in the dark.

### Room light (optional)

Press the room light button  **7** briefly.

► Room light is turned on or off.

### Reading light (optional)

Press the reading light button  **18** briefly.

► Reading light is turned on or off.

## Menu functions

### Set display brightness

1. Press button **MENU** **5**.
2. Select **Brightness** on the display **12** using the program selection buttons **P+ / P-** **14**.
3. Confirm with **OK** button **15**.
4. Set the desired brightness using **- / +** buttons **4**.
5. Using the menu button **MENU** **5** to leave menu.

### Set display contrast

1. Press button **MENU** **5**.
2. Select **Contrast** on the display **12** using the program selection buttons **P+ / P-** **14**.
3. Confirm with **OK** button **15**.
4. Set the desired contrast using **- / +** buttons **4**.
5. Using the menu button **MENU** **5** to leave menu.

### Set volume

1. Press button **MENU** **5**.
2. Select **Volume** on the display **12** using the program selection buttons **P+ / P-** **14**.
3. Confirm with **OK** button **15**.
4. Select for which function the volume should be set using the program selection buttons **P+ / P-** **14**.
5. Set the desired volume using **- / +** buttons **4**.
6. Using the menu button **MENU** **5** to leave menu.

# Optional functions

## TV

### Turn TV set on and off

Press TV button  ③ to turn the TV set on or off.

- ▶ The TV set is turned on and the last channel watched will be indicated on the display ⑫.

### Choosing a channel


Switch to the next channel using the program selection buttons **P+ / P-** ⑭.

### Setting sound and volume

Sound output via the loudspeaker ⑪ or connected headphones (connection socket ⑨).

Set the desired volume using **- / +** buttons ④.

### Call up teletext

1. Press teletext button  ⑰ to call up or leave teletext mode.
2. Switch to the next page using the program selection buttons **P+ / P-** ⑭.

## Radio

### Turn radio on and off

Press radio button  ⑬ to turn the radio on or off.

- ▶ The radio is turned on and the last radio channel listened to will be indicated on the display ⑫.

### Choosing a channel


Switch to the next channel using the program selection buttons **P+ / P-** ⑭.

### Setting sound and volume

Sound output via the loudspeaker ⑪ or connected headphones (connection socket ⑨).

Set the desired volume using **- / +** buttons ④.

## Blinds control

1. Press special functions button  ⑯.
2. Select **Blinds** on the display ⑫ using the program selection buttons **P+ / P-** ⑭.
3. Open or close the blinds using the **- / +** buttons ④.

## Infrared receiver

The infrared receiver is used to remotely control the patient terminal via environmental control devices.

<b>1 / 2</b>	<b>Call buttons</b>
<b>3</b>	<b>TV button</b>
<b>4</b>	<b>-/+ button</b>
<b>5</b>	<b>Menu button</b>
<b>6</b>	<b>Service call button with LED (finder light)</b>
<b>7</b>	<b>Room light</b>
<b>8</b>	<b>Infrared receiver (optional)</b>
<b>9</b>	<b>Headphone connection</b>
<b>10</b>	<b>Microphone</b>
<b>11</b>	<b>Loudspeaker</b>
<b>12</b>	<b>Display</b>
<b>13</b>	<b>Radio button</b>
<b>14</b>	<b>Program selection buttons</b>
<b>15</b>	<b>Confirmation button</b>
<b>16</b>	<b>Special function button</b>
<b>17</b>	<b>Teletext button</b>
<b>18</b>	<b>Reading light</b>
<b>19</b>	<b>LED of light button (finder light)</b>



**CAUTION**

**Do not use as rising aid**

Cable and device are no rising aid. Do not pull yourself up using the device.



SCHRACK SECONET AG  
Eibesbrunnnergasse 18  
A-1120 Vienna  
Tel. +43 1 81157  
[office@schrack-seconet.com](mailto:office@schrack-seconet.com)  
[www.schrack-seconet.com](http://www.schrack-seconet.com)

EN

HEALTH CARE

**SCHRACK**  
SECONET